



HELIX TEST SYSTEM

REF 017168

Dispositivo per verificare la corretta penetrazione del vapore nelle autoclavi di Classe B, in corpi cavi.

Non utilizzare il tubo per più di 250 test.

Conforme a EN ISO 11140-1, EN 867-5 Classe 2.

Istruzioni per l'uso:

1. Controllare che il dispositivo sia integro, asciutto, non danneggiato e non caldo.
2. Aprire il tappo e rimuovere la capsula del dispositivo.
3. Posizionare una striscia all'interno della capsula, assicurandosi che non si muova.
4. Chiudere il dispositivo.
5. Avviare il ciclo di sterilizzazione secondo i parametri indicati: per 3,5 minuti a 134°C oppure per 15 minuti a 121°C.
6. Al termine del ciclo, rimuovere il dispositivo caldo dall'autoclave, maneggiandolo con cautela per evitare ustioni.
7. Aprire la capsula e rimuovere l'indicatore per verificare il risultato, in base ai parametri indicati sulla confezione. Il test è positivo solo se tutte le barre hanno virato completamente al nero. Un viraggio non uniforme indica la presenza di aria residua nella macchina e, quindi, un possibile malfunzionamento dell'autoclave o un'esecuzione non corretta del test.

Legenda simboli:



Monouso



Conservare sigillato in un luogo asciutto



Non contiene piombo



Conservare a temperatura ambiente



Riciclabile



Evitare l'esposizione alla luce diretta



HELIX TEST SYSTEM

REF 017168

Device for verifying proper steam penetration in Class B autoclaves, in hollow loads.

Do not use the tube for more than 250 tests.

Compliant with EN ISO 11140-1, EN 867-5 Class 2.

Instructions for use:

1. Check that the device is intact, dry, undamaged, and not hot.
2. Open the cap and remove the capsule from the device.
3. Place the strip inside the capsule, ensuring it does not move.
4. Close the device.
5. Start the sterilization cycle according to the parameters on the indicator: 3,5 minutes at 134°C or 15 minutes at 121°C.
6. At the end of the sterilization cycle, remove the hot device from the autoclave, handling it carefully to avoid burns.
7. Open the capsule and remove the indicator to check the result, according to the parameters indicated on the packaging. The test is considered positive only if all bars have fully turned black. An uneven colour change indicates the presence of residual air in the autoclave, possibly due to a malfunction or incorrect execution of the test.

Key symbols:



Single-use



Store sealed in a dry place



Does not contain lead



Store at room temperature



Recyclable



Avoid exposure to direct light